

New membrane for *simpliwet®* easy!

simpliwet® Permanent membrane (No. 0038 round and 0058 master)

The filter membrane for *simpliwet®* unlike other moisture maintaining systems is different from existing filter membrane systems. It offers a very smooth working surface, similar to a glass plate. It inhibits the separation of filter fibres even after long term usage and the permanent membrane is printed with pre-arranged sections, which can be marked individually with water proof ink pens.

Note: do not scratch and mix with metallic instruments onto the surface of permanent membrane!

simpliwet® permanent membrane and *simpliwet®* liquid offer a great working combination.

The permanent membrane, in conjunction with *simpliwet®* liquid (Art. 0060) can be used over a long period of time and the *simpliwet®* liquid is sanitised and helps prevent germs.

The permanent membrane can be cleaned carefully with soap water and a brush or with a steam cleaner.

simpliwet® permanent membrane does not replace the used *simpliwet®* filter system.

It should be noted that the warm humid air in a lab could very likely generate germs – and these germs could also get on your *simpliwet®*. Therefore it is recommended that you should heat the honeycombs on a regular interval like every other week (you can place the *simpliwet®* honeycombs, while wet, into the preheated oven) as well as replace or clean the filter. This will help to insure that you're working with sanitised conditions in your lab.