


simpliwet® - Porcelain Tray – for ceramic professionals


simpliwet® round

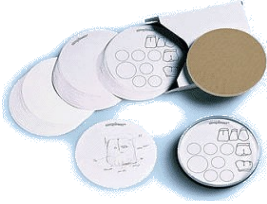


simpliwet® benefits

- even and continuously moisturized ceramic powder
- perfect saturation ratio
- speedy and swift working
- excellent layer-to-layer bonding
- no oxygen layer
- hygienic (can be sterilized at up to 1000° C)


simpliwet® master

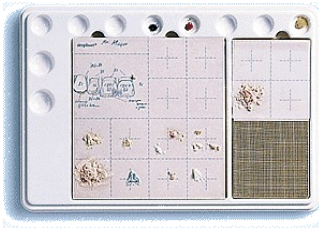

Fill **simpliwet** modelling fluid (0060) or distilled water (ca. 50ml) in the container and place the honeycomb base inside. After one minute, drain excess water. Place the filter paper or the permanent membrane on top of the honeycomb ceramic disc. Wait until the filter membrane is wet. Your **simpliwet** is now ready for use. The ceramic you apply saturates itself to a working consistency and remains saturated for several days. Add the glass cover and store your **simpliwet** in a refrigerator during longer breaks. You can make notes or draw diagrams of the shade build-up on the paper. After finishing the paper can be dried and filed for future reference.

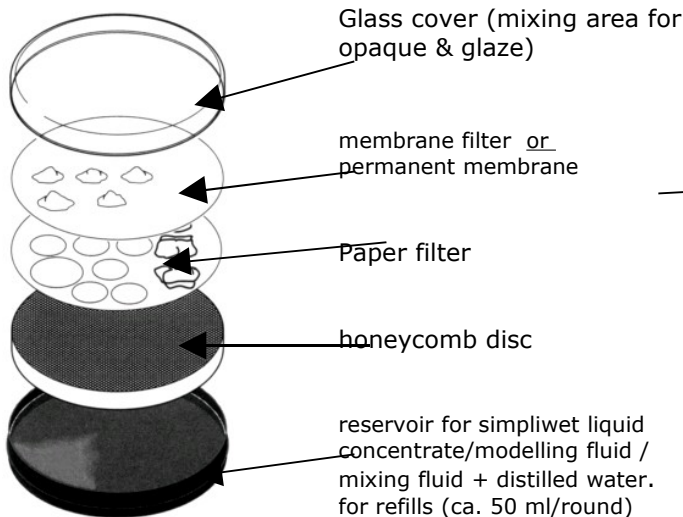

simpliwet easy

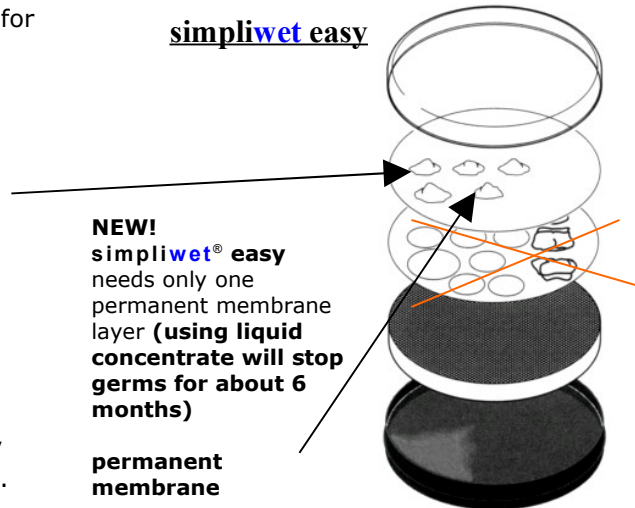

simpliwet® round (Art. No. 0030)

Delivery form:

- 40 paper filter sheets, round, blanc/printed
- 20 membrane filter sheets round
- 1 container 120 x 17mm
- 1 honeycomb disc, round 118x11mm
- 1 glass cover 120 x 15mm
- sufficient for ca. 50 ml liquid –

simpliwet® round easy is delivered with one permanent membrane.(Art. No. 0030E)
product is patent pended

simpliwet® master Art. No. 0050)

Delivery form:

- 50 paper filter sheets
- 25 membrane filter sheets / 3 printed filter sheets
- 1 container 300 x 200 x 40mm
- 1 honeycomb disc 150 x 150 mm
- 1 honeycomb disk 150 x 75 mm
- sufficient for ca. 250 ml liquid –

simpliwet® master easy is delivered with one permanent membrane. (Art. No. 0050E)

K.Fleischfresser Dentalerzeugnisse

Birkenau 17 • 81543 München / Germany

Fon +49(0)89 18944 507 • Fax +49(0)89 18944 508 • mobil +49(0)172 7963921